

How To Use RAPIDSTOP Tourniquet



Easy as 1-2-3 Three simple and fast steps set RapidStop® Tourniquet apart from the competition. Included are instructions on the product, pocket-sized instruction cards and video format. Integrated instructions on the RapidStop® Tourniquet are included using stitched waterproof vinyl tags resistant to any wear and tear so you will always know how to use RapidStop Tourniquet.

An individual who has sustained an injury to the arm or a leg and fully severed either the brachial, radial or ulnar artery has a window of approximately 60 seconds to stabilise the limb before decreasing blood pressure renders the individual unconscious. Shock can set in even sooner. Achieving full occlusion in the shortest time possible is critical. Faster application means the injured are stabilised sooner and/or more victims can be saved.

1

PULL D-RING FIRMLY UNTIL SNUG



2

CRANK LEVER REPEATEDLY UNTIL BLEEDING STOPS



3

RECORD TIME, WRAP & TUCK EXCESS STRAP

