

Fitting Instructions



1. Hold the respirator in your hand with the nosepiece at your fingertips allowing the headbands to hang freely below your hand.

2. Position the bottom of the respirator under your chin and the nosepiece against the bridge of your nose.



3. Pull the bottom strap over your head and place it around the neck and below the ears. Pull the top strap over your face and position it high at the back of the head.

4. Use both hands to carefully mould the nosepiece to the shape of your nose.



5. Perform a negative pressure fit check by completely covering the respirator with both hands and inhaling sharply. Mask should sink into face indicating an adequate seal.

6. If leaks are detected around the nose or edges adjust nose clip or straps as necessary to obtain an adequate fit. DO NOT enter the contaminated area if you cannot achieve a proper fit.